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Wydanie EJ 08.2022

Simply Bubble

Anti-bedsore alternating pressure bubble mattress

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PLEASE READ WHOLE INSTRUCTION
CAREFULLY BEFORE USE.

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Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual

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Alternating pressure bubble anti-bedsore mattress equipped with end flaps which allows to place the mattress on the bed. Maximum weight capacity: 120 kg

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Prevention and support of treatment of stage I and II bedsores.

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In case of adverse skin changes you should immediately consult a physician or medical personnel.

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- Place the mattress on a standard bed so that the air intake would be located on the side of patient's legs. Put the end flaps under the bed mattress.
- Check if the switch is in the "off" position.
- Connect the pump to electricity.

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- Connect the pipes to the pump – insert them into the air outlet located on the pump side (Illustration 4.) and on the mattress (Illustration 5.).
- Place a pump on hard surface or hang it on hinges. The place where the pump will stand must be clean and dry. Dust, hair or other impurity may block the filter.
- Please, check if the pipes are not folded or pressed.
- Switch on the pump.
- Set the pressure regulator on maximum. It will shorten the time of pumping which usually takes 20-30 min.
- After inflating the mattress, you should choose the pressure according to the doctor's recommendation.
- Place a patient on the mattress.

In order to receive exact instructions concerning an appropriate use and optimal pressure, please, consult your doctor. The mattress and pump create effect of "constant movement" (changing position of body support points), provide constant muscle massage and improve blood circulation. Thus, the mattress is an effective tool to prevent bedsores. But the position of the patient on the mattress needs to be changed despite of using it.

Switching off

- Turn the switch into the "OFF" position.
- Disconnect the pipes from the air outlet in order to prevent air from escaping.

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Illustration 1 shows the range of adjustment mattress inflating located on the pump. Depending on your preferences, please choose the right degree of firmness on a scale of 1 SOFT to 5 FIRM.

The mattress is inflated by a system of pipes. The cells are alternately inflated which ensures changing position of body support points. Patient's body rests on the same, invariable height regardless of the phase of pump functioning.

Phases of pump functioning:

1 phase: Inflating all the cells

2 phase: Deflating the first section of cells

3 phase: Reinflating all the cells

4 phase: Deflating the second section of cells

The phases occur in cycles every 10-15 minutes.

PROBLEM SOLVING

- Do not try to repair the pump, do not open it as it can lead to electric shock!
- The knob is in "ON" position but the pump does not work.

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- Please, check if the cable is plugged to electric supply and to the pump. If not, you should plug it.
- Please, check if there is a voltage in the electric circuit. If there is not, find the cause of such situation.
- Please, check if the cable is not damaged. If so, please contact the manufacturer.
- The pump works too slowly or the pressure is too low.
- Please, check if the pressure regulator is set on an appropriate pressure. If not, please, regulate it.
- Please, check if the pipes are correctly connected to the air outlet. If not, press them in such a way which prevents the air from escaping.
- Please, check if the pipes are not folded or pressed. If so, please, unbend them.
- Please, check if the cells are not damaged. If so, please, repair it using the reparation kit.
- Please, check if the pump works correctly – if you can hear some strange sounds like hissing etc., please contact the manufacturer

CARE AND MAINTENANCE

- Disconnect the pump from electricity. Clean all the surfaces with a moist cloth. Before switching on it has to be dry.
- The mattress should be cleaned with water with soap. After cleaning, dry it carefully. The mattress mustn't be washed in high temperature, dried in a washing machine, bleached, chemically washed, ironed or pressed.

The mattress may be disinfected using a product Incidin Plus, in accordance with the instructions for use of the preparation.

You must not use:

- solvents,
- toilet detergents,
- sharp brushes and hard objects,
- detergents that contain chlorine,
- corrosive detergents,
- Do not clean the exerciser with pressure or steam aggregates!

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Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging.	Check if any elements have any visible mechanical damage.
Once a month	Cleanliness and general condition.	In case of mechanical damage contact the producer immediately.

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Repair

- In case of reparation, please contact the producer.

Customer service

- If you have any questions or need help, contact the producer.

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Pump:

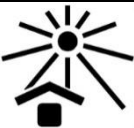
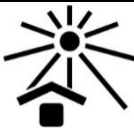
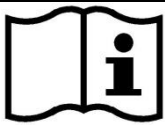
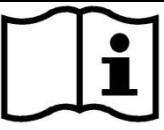
Power supply:220/240V 50Hz
 Air output: 6-7 liter/min.
 Pressure range: 70 mmHg – 130 mm Hg
 Cycle time: 10-15 min
 Power consumption: 7 W
 Size: 240 mm x 110 mm x 80 mm
 weight: 1kg
 Pump volume: 30-35 dB

Mattress:

Size : 2000mm x 900mm x 70mm
 Weight: 2,2 kg
 material: PCV
 colour: nude

Series no.: on the packing
 Durability: 3 years from production date.

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The product is compliant with standards on electrical and electronic equipment. It should not be disposed with other waste from households. The user is responsible for providing the equipment at the end of its service life to the appropriate collection points, under penalty of sanctions provided by the law applicable in the matter of waste disposal. For further information on the collection schemes available can be obtained from the local waste disposal centre.

10. Dane teleadresowe / Contact details

mdh Sp. z o.o.
ul. Maratońska 104, 94-007 Łódź, Poland
tel. +48 42 674 83 84
fax. +48 42 636 52 21
www.mdh.pl www.viteacare.com